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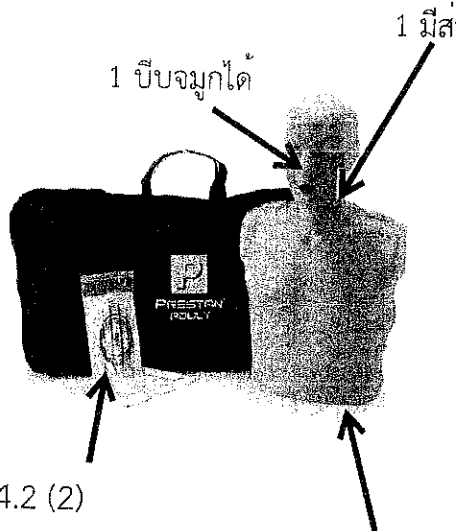
PRESTAN ADULT MANIKINS

SKU: PP-AM-100M-MS

The PRESTAN Professional Training Manikin series is unlike any other on the market. Uniquely designed for realistic experiences, they are available individually or in sets of four for group training. CPR manikins are designed to match the real-life CPR experience.

Features:

- Fully compliant with current industry guidelines, including the most recent AHA Integrated Feedback Directive.
- Realistic looking and performing manikin is more acceptable to both instructors and students.
- Far easier and faster to carry and set up at training sites.
- Patented face/head tilt simulates how an actual victim's head would move if he required CPR and teaches students how to properly open the airway.
- Easily inserted face-shield/lung-bag provides visual chest rise to ensure students receive the most accurate and realistic training.
- The face shield is secured so it won't slip and there's no need for adhesives.
- Manikins are available with the CPR Feedback Monitor - Monitors may also be purchased separately to easily install into non-monitored Prestan Professional Manikins.
- Prestan CPR Manikins are not made with natural rubber latex.

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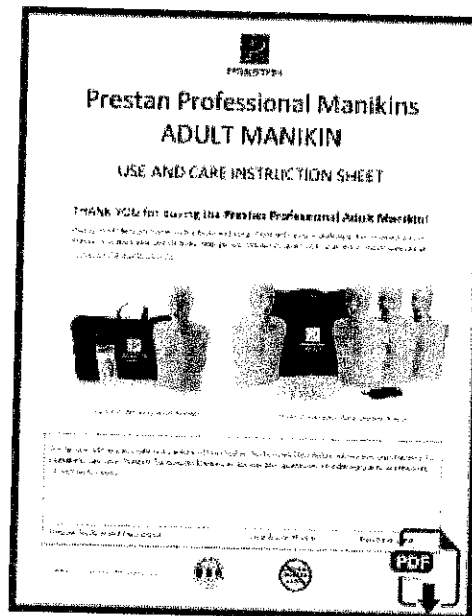
ADULT MANIKIN

USE AND CARE INSTRUCTION SHEET

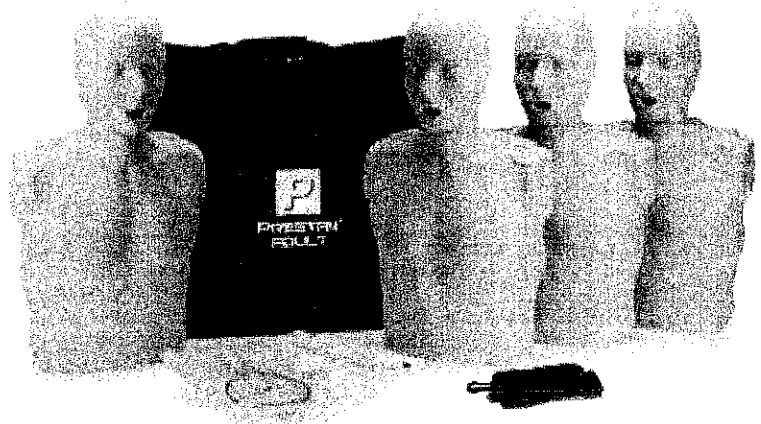
THANK YOU for buying the **Prestan Professional Adult Manikin!**

Please read through these instructions and keep them with your manikin(s). For information on Prestan's Authorized Distributors, help guides, resources and more, visit the Prestan website at www.prestanproducts.com.

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Prestan Professional Adult Manikin

Prestan Professional Adult Manikin 4-Pack

For future reference, make note below of the Prestan Authorized Distributor where you purchased your manikins. Call your Prestan Authorized Distributor for product questions, reordering parts/accessories, or warranty issues:

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Distributor Phone

Purchase Date

Prestan Professional ADULT MANIKINS

MODELS COVERED BY THIS INSTRUCTION SHEET

PRODUCT # (MODEL)	MANIKIN PACK	MANIKIN QTY/SIZE	INCLUDED CPR RATE MONITOR(S) *	INCLUDED CARRY CASE	INCLUDED FACE-SHIELD/ LUNG-BAGS
PP-AM-100M PP-AM-100M-MS PP-AM-100M-DS	SINGLE	1 ADULT	YES	SINGLE CASE	10-PACK
PP-AM-100 PP-AM-100-MS PP-AM-100-DS	SINGLE	1 ADULT	NO	SINGLE CASE	10-PACK
PP-AM-400M PP-AM-400M-MS PP-AM-400M-DS	4-PACK	4 ADULTS	YES 1 PER MANIKIN	4-PACK CASE w/strap	50-PACK
PP-AM-400 PP-AM-400-MS PP-AM-400-DS	4-PACK	4 ADULTS	NO	4-PACK CASE w/strap	50-PACK
PP-FM-300M PP-FM-300M-MS PP-FM-300M-DS	COLLECTION **	1 ADULT, 1 CHILD, 1 INFANT	YES 1 PER MANIKIN	COLLECTION CASE w/strap	10-PACK PER MANIKIN
PP-FM-500M PP-FM-500M-MS PP-FM-500M-DS	FAMILY PACK **	2 ADULTS, 1 CHILD, 2 INFANTS	YES 1 PER MANIKIN	FAMILY PACK CASE w/strap	10-PACK PER MANIKIN

REGARDING THE ABOVE CHART AND THE CONTENTS OF YOUR ORDER:

* The **CPR Rate Monitor** (*Rate Monitor, CPR Monitor, Monitor*) is a patented feedback feature for **Prestan Professional Manikins** that consists of an LED light bank that responds to compressions (see the section on the CPR Rate Monitor). The CPR Rate Monitor (*requires two (2) 'AA' batteries; not included*) is pre-installed in the left shoulder of the adult manikin. If you purchased the Prestan Professional Manikin without the CPR Rate Monitor and later decide you would like one, it may be ordered separately from your Prestan Authorized Distributor and easily installed. Installation instructions may be found in the Resources section of www.prestanproducts.com.

** The **Prestan Collection** and **Prestan Family Pack** also include **Prestan Professional Infant and Child Manikins**. These have separate Instruction Sheets that are included with the manikins or can be accessed at www.prestanproducts.com.

Available Skin Tones: Any of the above models may be available in one of three Skin Tones:

- A model number without a suffix signifies our original Light Skin Tone
- A model number with a "-MS" at the end (i.e. PP-AM-100M-MS) signifies a Medium Skin Tone
- A model number with a "-DS" at the end (i.e. PP-AM-400-DS) this signifies Dark Skin tone

Black Felt Dots are included on a strip for optional use with Adult Manikins. These may be adhered to the underside of the manikin torso to help prevent sliding on slippery floor surfaces.

If anything is missing, please contact the Prestan Authorized Distributor where you made your purchase.

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Prestan Professional ADULT MANIKINS

CLEANING

- The face and the body skin of the Prestan® Professional Manikin is made out of the same material and can be cleaned in the same manner.
- If the material becomes dirty, clean with a mixture of soap and water. If the area is extremely dirty, common household cleaners such as Formula 409®, Windex® or Vim® may be used. Non-bleach wipes such as Clorox® or Lysol® disinfecting wipes are convenient and work well. It is recommended that the use of these cleaners be done cautiously with minimum amount of abrasion to insure the life of the skin.
- A melamine foam eraser such as Mr. Clean® Magic Eraser® or Scotch-Brite™ Easy Erasing Pad may be used cautiously for stains.
- Cleaners containing wax, oil, or citrus (d-Limonene) are not recommended.
- Any cleaner not listed/recommended above may be tested in a non-visible area of the manikin. Open the manikin's torso chest and test the cleaner on the underside. Monitor for 24 hours. Should there be an undesirable reaction, the cleaner should not be used on the manikin.
- The face skin may also be sanitized with alcohol wipes or soaked in a solution of 1/4 cup bleach mixed with one gallon of water for 10 minutes. Between student uses, the instructor (or student) may clean the faces off with an alcohol wipe as an extra measure of safety.
- Don't forget to wipe the upper chest foam and skin in the same way in order to sanitize the inside of the manikin as well.
- Replacement skins for the face and torso may be purchased from your Prestan Authorized Distributor.

STORAGE

- Store the Prestan Professional Manikin(s) in the nylon carry bag(s).
- The manikin head should be removed for protection of the facial features, specifically the nose.
- For added protection, store the manikin inside the carry bag, and place the carry bag inside the original shipping box.
- Do not stack the manikins unless they are inside the original shipping box or other protective container.
- The nylon carry bag for the Prestan Professional Manikin(s) is not meant to store or transport belongings (i.e. books) in addition to the manikin(s) - please use the carry bag only for the intended manikin(s) to prevent damage to the bag.
- Store the manikin(s) in a cool, dry area, at temperatures between 50°-70° F and 50% relative humidity.
- For storage longer than 1 month, it is recommended that the batteries be removed from the CPT Rate Monitor (if the manikin has this feature).

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Prestan Professional ADULT MANIKINS

CPR FEATURES

FACE-SHIELD/LUNG-BAG FOR THE VISIBLE CHEST RISE

- The Face-Shield/Lung-Bag (PP-ALB-10, PP-ALB-50) combines the Face Shield and the Lung Bag into a single, t-shaped airway for the manikin. Air goes in through the Face-Shield opening to inflate the Lung-Bag, which provides the visible chest rise in response to administered CPR breaths.
- Face-Shield/Lung-Bags are sized specifically to fit each Prestan Professional Manikin (Adult/Child/Infant) and are not interchangeable among the different manikin sizes.
- The Face-Shield portion covers the mouth of the manikin. It stays in place by hooking the ends onto buttons located at each side of the head under the ears.
- The Lung-Bag portion is placed on the manikin chest (under the torso skin) and fills with air from administered CPR breaths. This provides a visual chest rise for the student.
- Face-Shield/Lung-Bags are a consumable item and should be changed depending on how they are used:
 - If breaths are being administered by mouth, either directly or via rescue mask adaptor, the Face-Shield/Lung-Bag should be changed between classes due to accumulation of bacteria-causing moisture inside the neck and bag. This change is advised for both sanitary and performance purposes.
 - If breaths are being administered by a manual resuscitator (BVM/Ambu bag), there is no need to change the Face-Shield/Lung-Bag.
- When a manikin is being shared by more than one student, it is recommended that each student be provided with their own sanitary and disposable mouth protector:
 - If direct mouth contact is being used to give CPR breaths, students should be supplied with their own Face Shield (PP-AFS-50) (sold separately and disposable). The fitted Face Shield can be placed over the already-installed Face-Shield/Lung-Bag on the mouth of the manikin when it is that student's turn to give breaths. Generic, non-fitted face shields may also be used, but may slide around on the mouth and possibly cover the mouth opening if not held in place.
 - If rescue masks are being used to give CPR breaths, students should be supplied with their own Rescue Mask Adaptor (10076-PPA, 10076-PPA-50) (sold separately and disposable). The adaptors can be inserted into any standard mask when it is that student's turn to give breaths.

CPR CLICKER

- The Clicker is the round disk in the center of the manikin's back.
- The Clicker works with compressions to provide an audible "click" when compression depth reaches 2" (the compression is able to continue to a 2.4" depth).
- The Clicker sends a signal to the CPR Rate Monitor (if the manikin has this feature) to allow it to provide feedback on compression speed (see CPR Rate Monitor – Guide)

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CPR RATE MONITOR - GUIDE

The CPR Rate Monitor (requires two (2) 'AA' batteries; not included) is a patented feedback feature that comes pre-installed in the shoulder area of the Prestan Professional Adult Manikin. If you have purchased a Prestan Professional Adult Manikin without the CPR Rate Monitor and later decide you want this feature – it may be purchased and installed separately by contacting your Prestan Authorized Distributor.

The Prestan Professional Training Manikin CPR Rate Monitor has a unique way of monitoring both the depth of compressions and the rate of compressions:

At the first compression all four of the lights in the upper left shoulder will come on, 4.1 indicating that the lights are working. (Figure 1)

If the lights do not light, make sure that the depth of compression is deep enough (you will hear a clicking sound). The lights will not work if the student is not compressing the manikin correctly.

Once a student begins chest compressions, the yellow light and the green lights will go out and, from zero to less than 60 compressions per minute, the red light will be on. (Figure 2)

4.2

As the student increases the rate of compressions and reaches 60 compressions per minute, a yellow light will come on until the student reaches 80 compressions per minute. (Figure 3)

As the student increases the rate of compressions and reaches 80 compressions per minute, the first green light will come on until the student reaches 100 compressions per minute. (Figure 4)

4.3

When the student reaches 100 compressions per minute, a second green light comes on, indicating that the recommended rate has been reached. (Figure 5)

If the student slows down, one of the green lights will go out indicating that the student needs to increase the rate of compressions. 4.4

Should compressions exceed 120 per minute, a yellow light will flash rapidly indicating the recommended cap has been reached. (Figure 6)

Always try to maintain the recommended speed between 100 and 120 compressions per minute as indicated by the 2 green lights. 4.5

If the student's depth of compression is not correct, at any time, all the lights will go off and the red light will blink.

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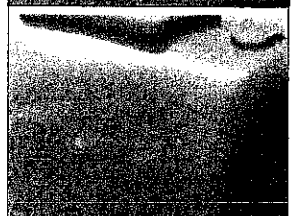
Figure 5; 100-120 cpm

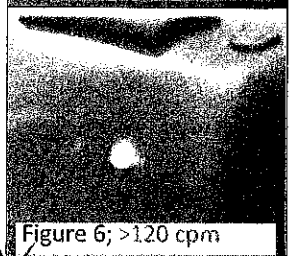
Figure 6; >120 cpm

Prestan Professional ADULT MANIKINS

FACE-SHIELD/LUNG-BAG INSTALLATION

1. Unlatch the torso of the Prestan Professional Manikin by popping the 'tabs' located at the top of the shoulders (Figure 1). Carefully lift the torso frame open by the neck.
Note: Do not pull on tabs (past popping the latch). Doing so may break the tabs or stretch out the frame over time, impairing the latching function.

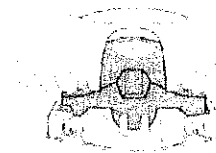
Figure 1

Figure 2

2. Attach the head to the torso by placing the neck of the head onto the two spindles on the neck portion of the torso (Figure 2).
3. Fold the Lung-Bag portion of the Face-Shield/Lung-Bag (Figures 3a-3d):
 - The Face-Shield/Lung-Bag has four 'dotted lines' printed on one side of the bag (along with an oval)
 - Using the four dotted lines, fold the Face-Shield/Lung-Bag into fourths, following the diagram below

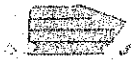
Figure 3a

Figure 3b

Figure 3c

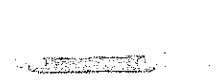
Figure 3d

4. Tilt back the head of the manikin (Figure 4).
5. Insert the folded Lung-Bag into the mouth of the manikin (Figure 5).
6. Push the Lung-Bag into the mouth until the bag exits through the neck of the manikin (Figure 6).
7. Grasp the Lung-Bag and finish pulling it through the neck of the manikin (Figure 7).
8. Attach the Face-Shield tabs to the 'buttons' located on the side of the manikin head (Figure 8).
9. Lightly pull down on the Lung-Bag to help open the mouth hole on the Face-Shield.
10. Then position the Lung-Bag on top of the compression plate (the hard plastic circular piece found on top of the torso foam core) with the circular imprint on the Lung-Bag aligning with the compression plate (Figure 9).
11. With proper positioning, the neck of the Face-Shield/Lung-Bag will have slack. Make sure it is not twisted.
12. Close the torso and secure the latch by pushing on the shoulder tabs until they snap into place.

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Prestan Professional ADULT MANIKINS

TROUBLESHOOTING GUIDE

SYMPTOM	POSSIBLE ISSUES OR COMMENTS	SOLUTIONS
CPR Rate Monitor		
Cannot find CPR Rate Monitor (Monitor) – no lights in the shoulder area	You have purchased a Prestan Professional Adult Manikin without the CPR Rate Monitor	If you were expecting a monitored manikin, check with the distributor where you purchased the manikin to insure you received what you paid for Note: CPR Rate Monitors may be purchased separately from your Prestan Authorized Distributor
CPR Rate Monitor (Monitor) lights are not working properly <i>To help understand how the Monitor works, please note that the Monitor lights are powered by a small current generated by the Clicker</i> <i>Please follow the troubleshooting tips at right in the order they are listed for best results</i>	- Chest compressions are not deep enough to signal Monitor - Batteries are not installed, need replacing or are not inserted correctly in the CPR Rate Monitor - The Clicker-Monitor connection is broken - Clicker may be clogged with plastic dust, interrupting the signal to the Monitor - Either the Clicker or the Monitor needs to be replaced	- Compress the chest until the click is audible from the Clicker at 2"-2.4" - Install or replace batteries into Monitor (back of manikin, left shoulder area) - requires 2-AA (1.5V) batteries 4.7 - On the manikin's back, remove the long skinny Wire Cover between the Clicker and Monitor – squeeze connectors to disconnect the wires then reconnect to insure Clicker-Monitor communications - Remove Clicker and blow out dust – then replace Clicker and reconnect with Monitor - Exchange Clickers and Monitors with working manikins (if available) to determine whether the Clicker (RPP-ACLICK1) or the Monitor (RPP-AMON-1) needs to be replaced*
CPR Rate Monitor (Monitor) lights are all on	Batteries are running low	Replace batteries
CPR Rate Monitor (Monitor) red light is blinking	This is normal after replacing batteries or manikin is paused - may blink up to 16 times to indicate ready status	Commence or resume compressions - blinking will stop and feedback rate signals will begin

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Prestan Professional ADULT MANIKINS

TROUBLESHOOTING GUIDE

(Continued)

SYMPTOM	POSSIBLE ISSUES OR COMMENTS	SOLUTIONS
Torso and Face-Shield/Lung-Bag		
Cannot hear a click when doing compressions	- Hand position and/or compression angle may be incorrect - Compressions may not be deep enough to activate the clicker - Clicker may be broken	- Confirm correct hand placement and compress straight down from directly above the manikin - Compress between 2 and 2.4 inches depth - Replace Clicker in back of manikin (RPP-ACLICK-1)*
Chest will not rise when breaths are given	- Head is not tilted back enough to open airway - Face Shield is not attached - Lung Bag may be damaged or torn (Note: there are holes in the Adult Lung Bag necks by design to allow deflation) - Lung Bag is not lying flat or properly positioned over the compression plate and/or neck of bag is twisted - Lung Bag has accumulated moisture in the neck, which may cause the sides to stick together and prevent air passage	- Push on forehead and/or lift at chin to tilt head back enough to open airway ๒ 1 ผัก head tilt - chin lift ได้ - Attach Face Shield at "buttons" on the side of the manikin's head below ears - Blow into the Face-Shield/Lung-Bag while it is outside the manikin to make sure it inflates properly – if not, discard and use another Face-Shield/Lung-Bag - Revisit instructions for installing a Face-Shield/Lung-Bag earlier in this manual - A new Face-Shield/Lung-Bag should be installed after each class, or more often if needed due to excessive moisture buildup
Manikin torso will not stay closed	Tabbed latches at shoulders may not be securely closed	Close tabs at shoulders – a snap will be heard when shoulder tabs are properly latched
Manikin torso opens when breaths are given	- Chest frame may be stretched from improper opening of torso - Tabbed latches at shoulders are broken from improper opening of torso	- Open torso and attempt to reverse stretched out frame by pushing it in the opposite direction สำเนาถูกต้อง - Torso Frame needs to be replaced (RPP-AFRAME-1)* (นางสาวสมลั พุ่มสุข)

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Prestan Professional ADULT MANIKINS

TROUBLESHOOTING GUIDE (Continued)

SYMPTOM	POSSIBLE ISSUES OR COMMENTS	SOLUTIONS
Torso		
Rattling in chest	Compression Sleeve may be broken	Replace sleeve in chest with part RPP-ACOM-1 (compression assembly includes chest plate, spring, sleeve and clicker)*
Chest Plate is broken	Typically results from a combination of incorrect (not centered) hand placement, compression angle not straight down from directly above and/or overzealous compression force	Replace chest plate with RPP-ACOM-1 (compression assembly includes chest plate, spring, sleeve and clicker)*

* Contact your Prestan Authorized Distributor to either purchase the replacement part or receive it at no charge if the manikin is within Prestan's 3-year warranty period and is a qualifying warranty repair.

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Prestan Professional ADULT MANIKINS

WARRANTY Three Year Limited Warranty

1. **WARRANTY COVERAGE:** Prestan offers a LIMITED WARRANTY against any defects in workmanship or materials under normal use with respect to its PRESTAN TRAINING MANIKIN.
2. **WARRANTY PERIOD:** Prestan warrants the PRESTAN TRAINING MANIKIN for three (3) years from the date of purchase. Any component of the PRESTAN TRAINING MANIKIN which proves defective with thirty-six (36) months from the date of purchase will be repaired or replaced, at Prestan's option, free of charge to the owner. The owner is responsible for shipping and handling fees to the place which Prestan designates.
3. **WARRANTOR'S RESPONSIBILITIES:** Prestan will repair or replace (at Prestan's option) a defective warranted component subject to the provisions of Section 2 without charge except for transportation costs. OWNER'S SOLE AND EXCLUSIVE REMEDY AGAINST PRESTAN SHALL BE FOR THE REPAIR OR REPLACEMENT OF DEFECTIVE GOODS AS PROVIDED HEREIN. Repair or replacement of any component of any PRESTAN TRAINING MANIKIN does not extend this limited warranty or begin a new limited warranty period.
4. **CIRCUMSTANCE NEGATING WARRANTY AND OTHER LIMITATIONS:**
 - a. Prestan reserves the right to refuse warranty service or replacement when product failure is due to misuse, mishandling, misapplication, accident, neglect, improper installation, modification, use of unauthorized parts or attachments, adjustments or repair performed by anyone other than Prestan or its authorized agent, or any causes other than defective workmanship or material.
 - b. This limited warranty does not cover abrasion, abnormal use, damage caused by mishandling, neglect or unauthorized modification. This warranty is void if the PRESTAN TRAINING MANIKIN is structurally altered.
 - c. Owner must provide proof of the date of purchase, place of purchase and the purchase price in order to obtain service under this limited warranty.
 - d. Prestan makes no warranty other than as contained herein. No agent, representative, dealer or employee of Prestan has the authority to alter the obligations or limitations of this limited warranty.
5. **LIMITATIONS OF IMPLIED WARRANTIES AND DAMAGES:**
 - a. Any and all implied warranties on the PRESTAN TRAINING MANIKIN, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXCEED IN DURATION THE RESPECTIVE TERMS OF THIS LIMITED WARRANTY WHICH BEGINS WITH THE DATE OF PURCHASE. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
 - b. PRESTAN SHALL NOT BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING OUT OF THE USE OF THE PRESTAN TRAINING MANIKIN OR THE BREACH OF THIS OR ANY OTHER EXPRESSED OR IMPLIED WARRANTY. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you. Any and all damages awarded for any breach of this limited warranty shall be limited to the amount of the purchase price.
6. **WARRANTY SERVICE.** To obtain warranty service contact your Prestan Authorized Distributor, or for information about Prestan, visit our website at www.prestanproducts.com.
7. **RIGHTS UNDER STATE LAW:** This warranty gives you specific legal rights and you may have other rights which vary from state to state.

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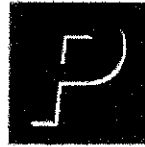
PRESTAN

Professional Quality. Trusted Worldwide.

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This is to certify that

PRESTAN CPR MANIKIN

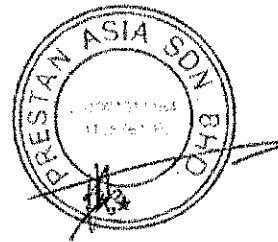
has been tested and demonstrated the ability to withstand more than **500,000** chest compressions under normal training conditions while maintaining its intended performance and functionality.

This certificate is issued upon request for official reference and procurement purposes.

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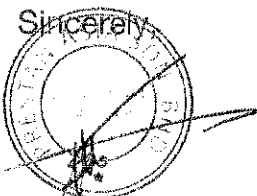
as our Authorized Distributor for the purpose of participating in the procurement
to Saraburi Provincial Administrative Organization

is one of our authorised distributor to sell/service, promote and market for all
Prestan CPR Manikins and related products in Thailand. This authorisation is valid
until 31 December 2026 and we hold to right to cancel the authorisation without
notifying. We also confirm that all Prestan products replacement parts will be
available for the next five years."

Please do not hesitate to contact us should you need further clarifications.

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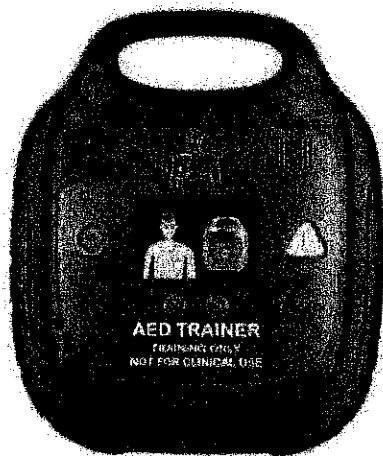
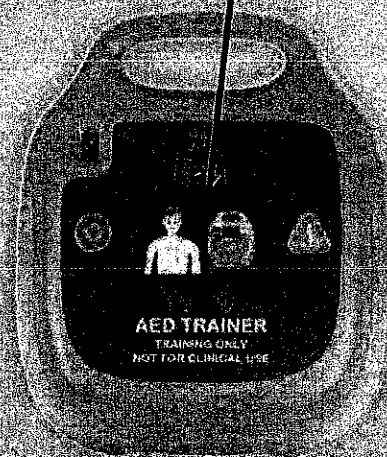
Product Introduction

The ATM-112 AED Trainer is designed to effectively train emergency responders using an Automatic External Defibrillator(AED). This training unit provides a variety of pre-programmed scenarios to help responders become familiar with an AED and allow them to demonstrate the basic skills necessary to use an AED in an emergency.

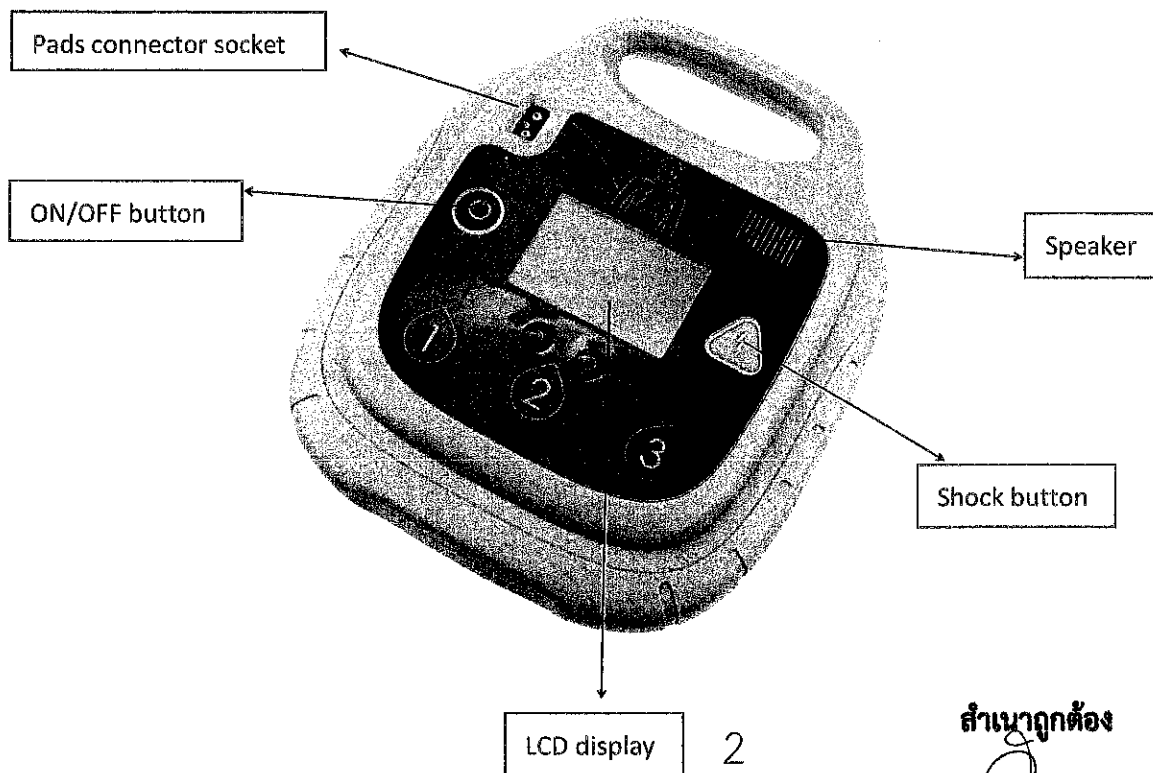
ATM-112 AED trainer provides simulated shock delivery, to ensure safety during training, it has no high voltage capabilities, and cannot deliver a shock.

NOTE: Before using a real clinical AED, the responder should have completed an appropriate responder training program. Having completed such a course, they should demonstrate a proficiency in performing Basic Life Support skills; and knowledge of applicable medical protocols and procedures.

Overview

Product Contents:

Main body, Remote Control, Training pads(Adult use 1pair. Pediatric use 1pair), Wire Connectors, instruction Manual, Storage Bag, Adapter.

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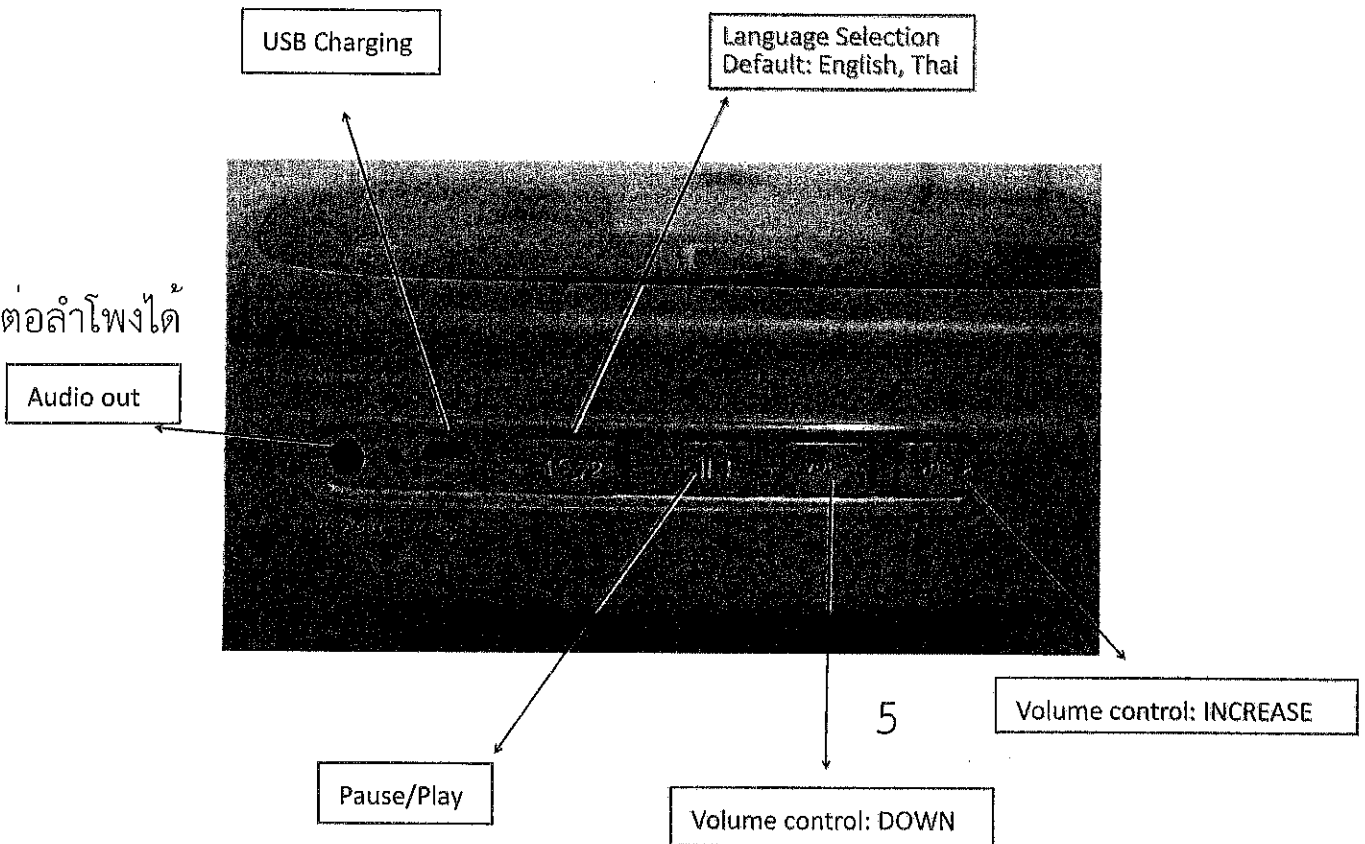
AED Trainer Main body

1. ON/OFF Button

Turn unit on and initiate voice prompts. Press once to turn on. Press again to turn off.

2. Pads Connector Socket

Insert the training pads connector into the pads connector socket. Both the flashing indicator light and the LCD screen indicate the socket location.

3. Volume Control

Control the voice prompt volume during use.

5

4. Shock button

Activate simulated shock delivery. The button&LCD screen flashes when the unit is ready to deliver a simulated shock.

5. USB Charging

Please use the ATM-112 AED Trainer certified adapter to charge the built-in Lithium Polymer Battery (3.7V /5000mAh).

(Note: Don't use the Trainer unit while adapter is connected)

6. Audio Out

All external speakers can be connected to the AED Trainer.

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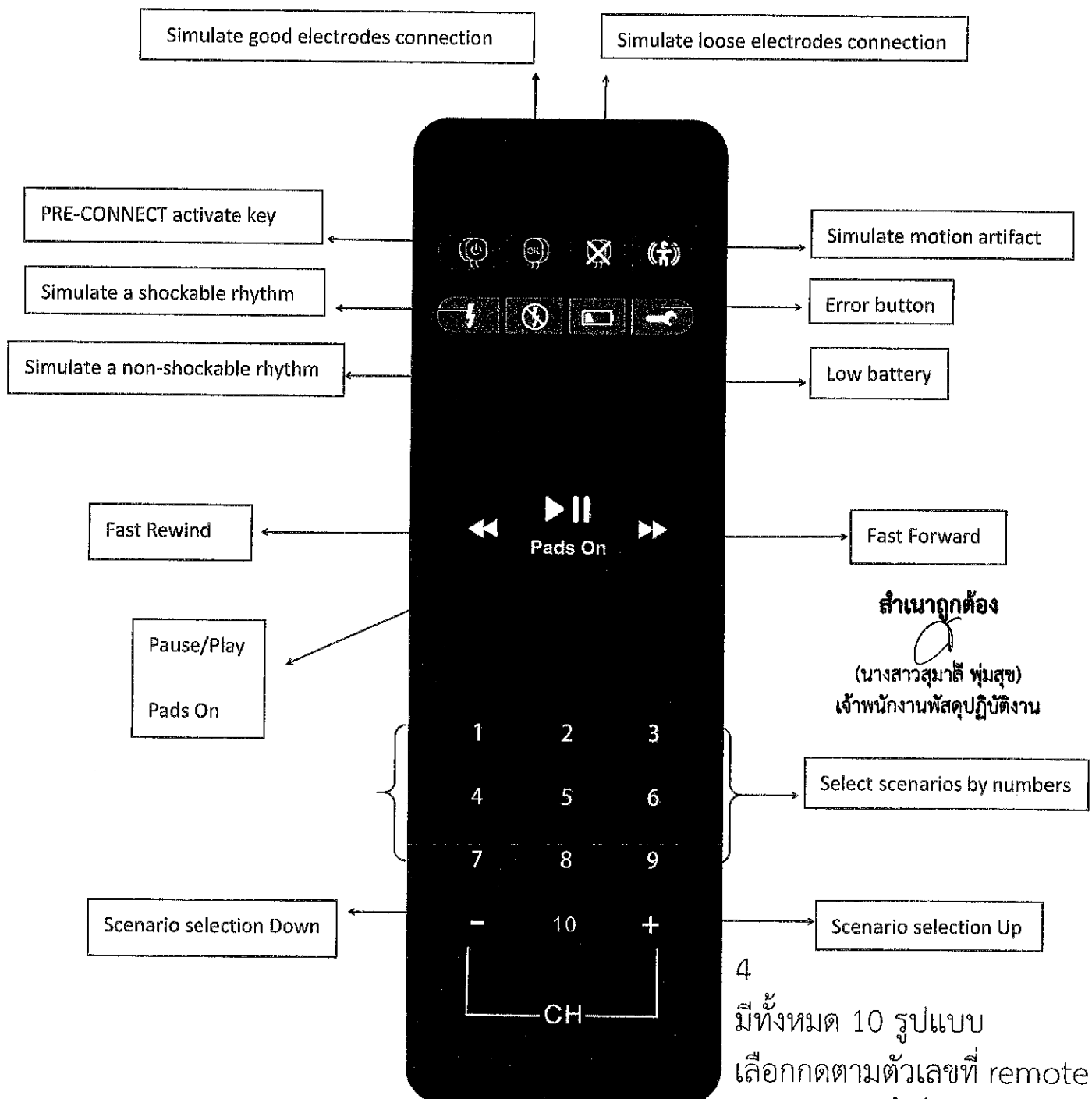
7. Pause / Play

The pause function can temporarily stop all operations of the unit. Press once to pause. Press again to continue playing.

Remote Control

3

Control buttons functions are described below (please refer to the diagram)

1. PRE-CONNECT ACTIVATE KEY

Teach students how to paste the electrodes and the cable. Simulate this process.
This is fully autoplay.

2. SIMULATE GOOD ELECTRODES CONNECTION

During the training, if this button is pressed, " " will be displayed on the LCD screen. It simulates good electrodes connection. The user can choose related scenarios to practice.

3. SIMULATE LOOSE ELECTRODES CONNECTION

During the training, if the button is pressed, " " will be displayed on the LCD screen. It simulates loose electrodes connection. The user can choose related scenarios to practice.

4. SIMULATE MOTION ARTIFACT

During the training, if this button is pressed, " " will be displayed on the LCD screen.

5. SIMULATE A SHOCKABLE RHYTHM

During the training, if this button is pressed, " " will be displayed on the LCD screen. It simulates a shockable rhythm. The user can choose related scenarios to practice.

6. SIMULATE A NON-SHOCKABLE RHYTHM

During the training, if this button is pressed, " " will be displayed on the LCD screen. It simulates a non-shock able rhythm. The user can choose related scenarios to practice.

7. LOW BATTERY

During the training, if this button is pressed, " " will be displayed on the LCD screen. A voice prompts "low battery", and then the current scenario is continued.

8. ERROR BUTTON

During the training, if this button is pressed, it gives "tick" sound, and " " will displayed on the LCD screen, the voice prompts are stopped, the "tick" sound is given again after 10 seconds and will be continue until the AED trainer is turned off.

9. PAUSE / PLAY

During the training, push the button will be stopped, push again will resumed.

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11. SCENARIOUS SELECTION / UP & DOWN

When either of these buttons are pressed, the display prompts a change in scenario selection.

Specification and Product Content

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
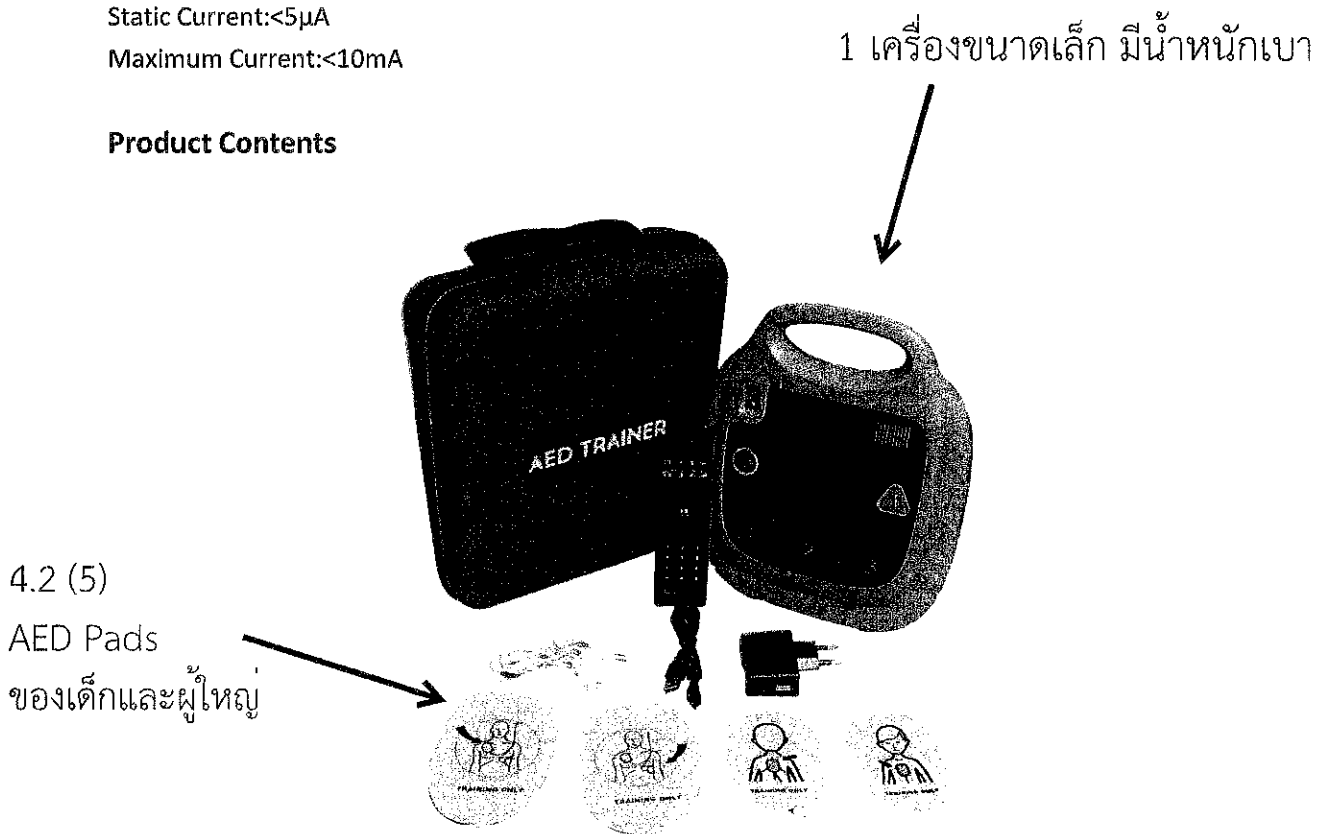
5	Child Pads	1	Pair	These pads can be used repeatedly
6	Instruction Manual	1	PC	4.2 (6)
7	Storage bag	1	PC	EVA bag
8	Adapter	1	PC	USB socket

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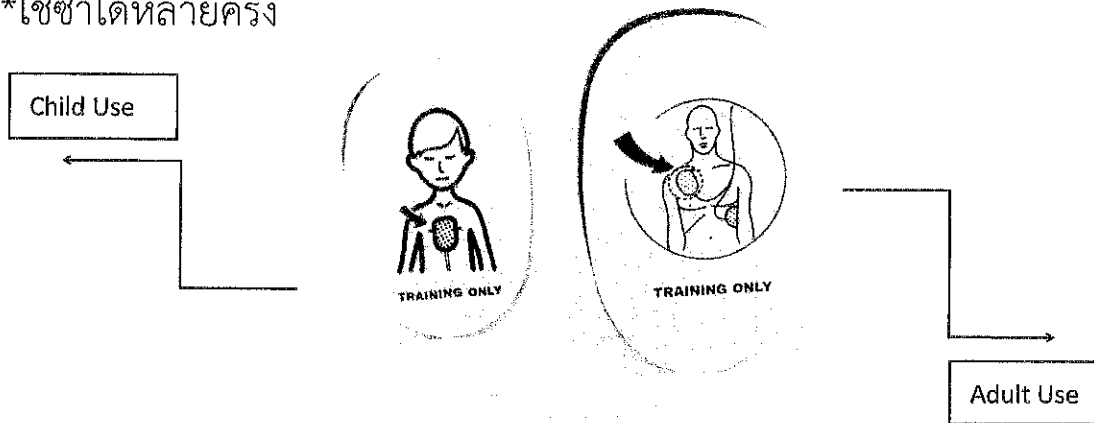Note: The training pads are for training use only and will not function with and actual clinical AED.

Operating Instruction

First, turn on the trainer and select a training scenario

Turn on the unit, then press the scenario selection button to select a preprogrammed training scenario, the operating scenario is displayed in the LCD display window.

Note: The last setting will be stored and maintained even if the unit has been turned off.

Second, connect pads

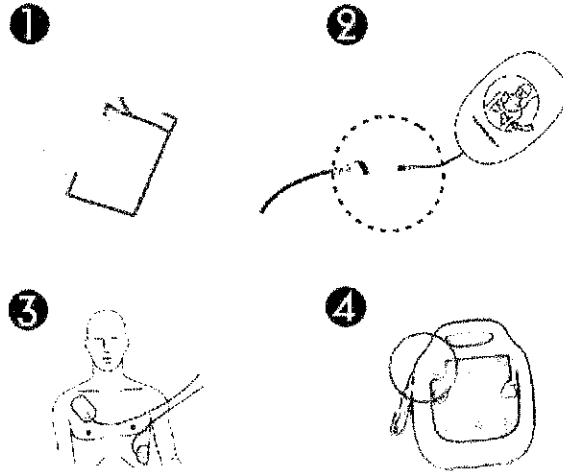
- Take out the training pads
- Connect the wire with training pads.
- Place pads on the chest in the position shown on the pad diagrams.
- When prompted, plug the connector to the AED trainer using the socket with a flashing Green light indicator (also a display about the socket location on the full colour screen to remind).

Note:

- Don't bend or stretch wires excessively.
- Please hold the pad connector when pulling it out of the trainer.

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
Third, follow the voice prompts

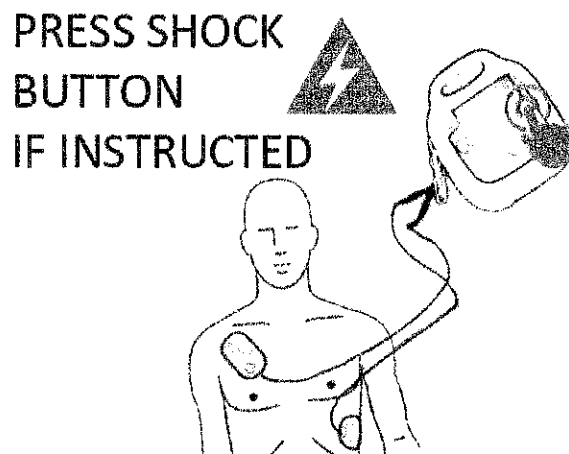According to the selected scenario, operate the trainer by following the voice prompts.

Storage Guidance

Pad Maintenance

- Keep the Gel side of pads clean and free of dust.
- After use, please stick pads with their original cover.
- If the stickiness of pads declines, users can try washing the pads with water for a few seconds, and then leaving to dry thoroughly.
- Don't wipe the Gel side of pads with paper and cloth.
- Don't scrape the Gel side of pads with nail and brush.
- The pads should last for several training sessions.
- Don't wash pads frequently, and don't use detergent and hot water.
- Don't store pads in places subjected to direct sunlight, high temperature or humidity.

Main Body Maintenance

- Wipe clean the main body routinely.
- After shutting off power, make sure to pull out the pad connector from the main body.

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- If the main body is dirty, use a soft damp cloth with water or neutral detergent and wipe softly. If necessary, clean or sterilize it with a cloth soaked in medical alcohol (75% concentration max).
- Do not use abrasive cleaning substances.
- Use only damp non-abrasive cloth for cleaning. Do not soak the AED training unit with any fluid.

Storage Precautions

- Do not store near spirits or gasoline.
- Do not store in locations subjected to direct sunlight, high temperature, moisture, dust or corrosive gas.
- Store the main body, electrode pads, wire leads and instruction manual in the bag provided.

* Failure to comply with the above instruction may void the warranty.

Pre-programmed Scenarios 1-10

Scenarios	Description	Operation
1	Defibrillation pads- With single-shock conversion	● Apply pads(pause) ● Pads on ● Shockable rhythm ● 1 shock ● CPR ● Non-shockable rhythm ● CPR (then repeats)
2	Ventricular Fibrillation- multiple shocks required	● Shockable rhythm ● 4 shocks ● CPR between shocks ● Non-shockable rhythm ● CPR (then repeats)
3	Ventricular Fibrillation- With single-shock conversion	● Shockable rhythm ● 1 shock ● CPR ● Non-shockable rhythm ● CPR (then repeats)
4	Ventricular Fibrillation- two shocks required	● Shockable rhythm ● 2 shocks ● CPR between shocks ● Non-shockable rhythm ● CPR (then repeats)
5	Non-shockable Rhythm	● Non-shockable rhythm throughout ● CPR (then repeats)
6	Ventricular Fibrillation- With 2 shocks required for conversion	● Shockable rhythm ● 2 shocks ● CPR between shocks ● Non-shockable rhythm ● CPR (Then the scenario starts)

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7	Ventricular Fibrillation- With 3 shocks required for conversion	● Shockable rhythm ● 3 shocks ● CPR between shocks ● Non-shockable rhythm ● CPR (then repeats)
8	Troubleshooting Skills- Defibrillation pads with two shocks required for conversion	● Poor pads connection ● Shockable rhythm ● 2 shocks ● CPR between shocks ● Non-shockable rhythm ● CPR (then repeats)
9	Ventricular Fibrillation	● Shockable rhythm ● 1 shock ● CPR ● Non-shockable rhythm ● CPR (Then the scenario restarts)
10	Troubleshooting Skills- Motion, low battery	● Detect motion ● Shockable rhythm ● 1 shock ● CPR ● Low battery ● Non-shockable rhythm ● CPR (then repeats)

Scenario 1:

Press pads firmly to patient's bare chest --Stop10S--

(Pause status) then If need continue, press Pads On button on remote control.

Analyzing heart rhythm --Stop2S--

Do not touch the patient --Stop5S--

Shock advised --Stop2S--

Charging --Stop2S--

Stay clear of patient --Stop2S--

Deliver shock now, press the orange button

(The shock light flashes, the speaker has an alarm sound and press shock button.)

(If shock button is pressed, then the voice will prompt:

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(Circulation)

(If shock button not pressed, the voice prompts: --Stop6S--

Deliver shock now, press the orange button --Stop6S--

Deliver shock now, press the orange button --Stop6S--

Shock button not pressed. --Stop2S--

Begin CPR.

--Stop 2MIN--

(Then the scenario restarts).

(But Press pads firmly to patient's bare chest. won't play.)

Scenario 2:

Analyzing heart rhythm --Stop2S--

Do not touch the patient --Stop5S--

Shock advised --Stop2S--

Charging --Stop2S--

Stay clear of patient --Stop2S--

Deliver shock now, press the orange button

(The shock light flashes, the speaker has an alarm sound and press shock button.)

(If shock button is pressed, then the voice will prompt:

Shock delivered --Stop2S--

Begin CPR.

--Stop 2MIN--

Analyzing heart rhythm --Stop2S--

Do not touch the patient --Stop5S--

Shock advised --Stop2S--

Charging --Stop2S--

Stay clear of patient --Stop2S--

Deliver shock now, press the orange button

(The shock light flashes, the speaker has an alarm sound and press shock button.)

(If shock button is pressed, then the voice will prompt:

Shock delivered --Stop2S--

Begin CPR.

--Stop 2MIN--

Analyzing heart rhythm --Stop2S--

Do not touch the patient --Stop5S--

Shock advised --Stop2S--

Charging --Stop2S--

Stay clear of patient --Stop2S--

Deliver shock now, press the orange button

(The shock light flashes, the speaker has an alarm sound and press shock button.)

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(If shock button not pressed, the voice prompts: --Stop6S--
Deliver shock now, press the orange button --Stop6S--
Deliver shock now, press the orange button --Stop6S--
Shock button not pressed. --Stop2S--
Begin CPR.
--Stop 2MIN--
(Then the scenario restarts).

Scenario 4:

Analyzing heart rhythm --Stop2S--
Do not touch the patient --Stop5S--
Shock advised --Stop2S--
Charging --Stop2S--
Stay clear of patient --Stop2S--
Deliver shock now, press the orange button
(The shock light flashes, the speaker has an alarm sound and press shock button.)
(If shock button is pressed, then the voice will prompt:
Shock delivered --Stop2S--
Begin CPR.
--Stop2 MIN--

Analyzing heart rhythm --Stop2S--
Do not touch the patient --Stop5S--
Shock advised --Stop2S--
Charging --Stop2S--
Stay clear of patient --Stop2S--
Deliver shock now, press the orange button
(The shock light flashes, the speaker has an alarm sound and press shock button.)
(If shock button is pressed, then the voice will prompt:
Shock delivered --Stop2S--
Begin CPR.
--Stop 2MIN--

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--Stop 2MIN--

(Circulation)

(If shock button not pressed, the voice prompts: Stop6S

Deliver shock now, press the orange button --Stop6S-

Deliver shock now, press the orange button --Stop6S--

Shock button not pressed. --Stop2S--

Begin CPR.

--Stop 2MIN--

(Then the scenario restarts:

(But --Stop2S-- Analyzing interrupted

--Stop2S-- Do not touch the patient. Won't play)

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Company

This AED trainer includes :

- 1 complete AED training unit with 10 BILINGUAL pre-programmed scenarios installed
- 1 remote control
- 1 set of Child pads
- 1 set of Adult pads
- 1 connector
- 1 adapter
- 1 manual
- 1 zippered EVA hand carrying case

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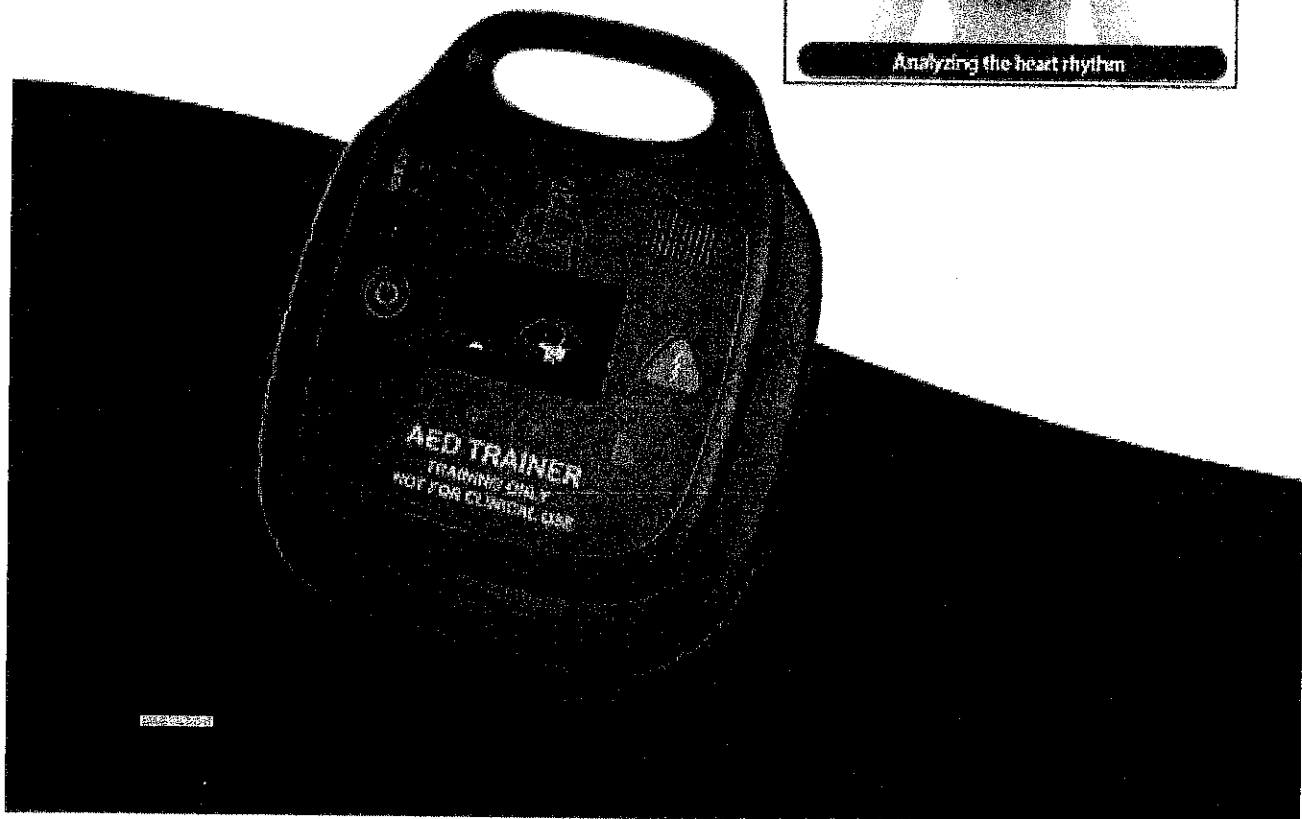
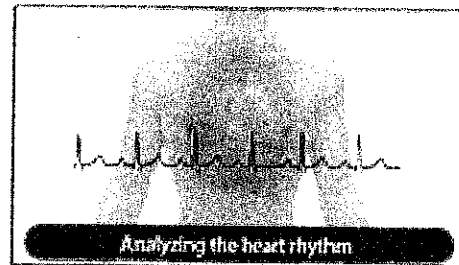
FULL COLOR SCREEN

Easy to study

Easy to study, the training unit has UI display working with the voice prompter

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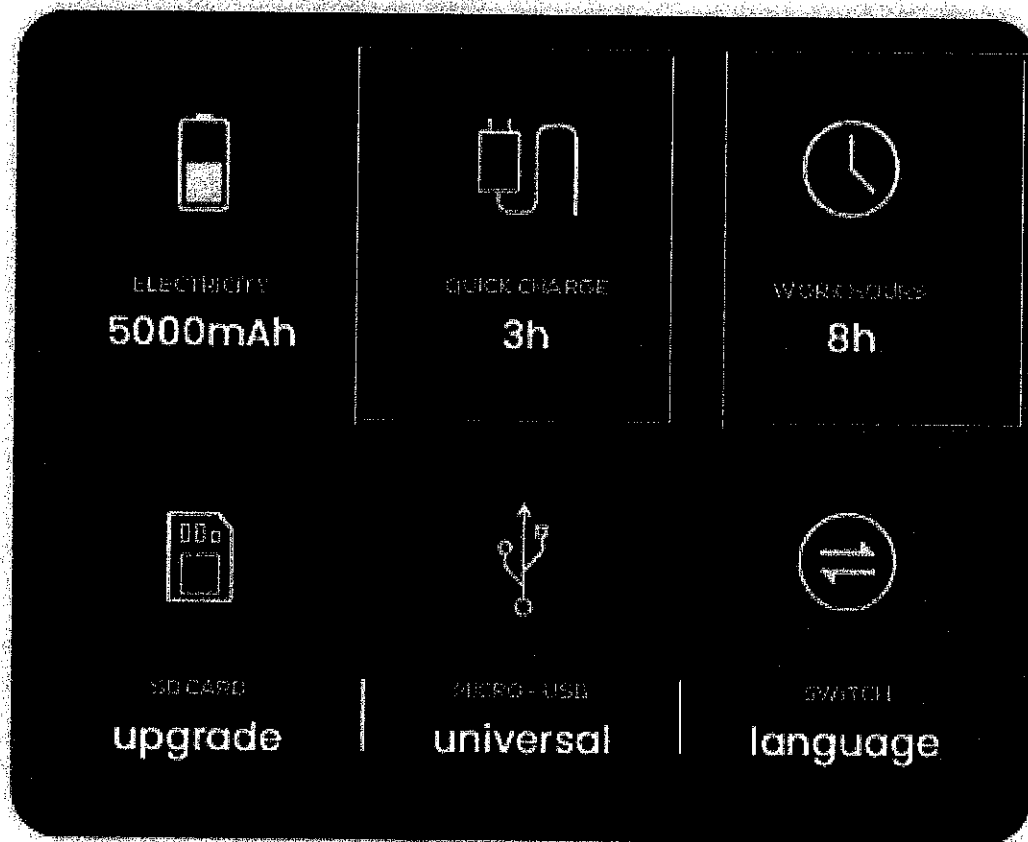
J4

RECHARGEABLE, UPGRADABLE

This unit is rechargeable and can be recharged with a standard 5V/1A power supply. The unit is also upgradeable and can be upgraded with a standard 5V/1A power supply.

8


7

RECHARGING

RECHARGING

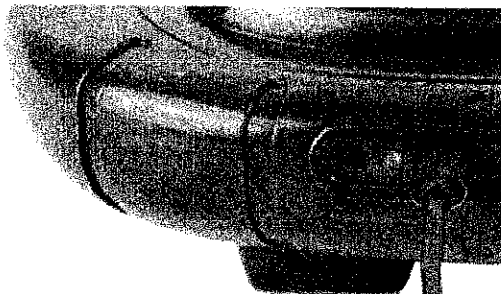This unit is rechargeable and can be recharged with a standard 5V/1A power supply. The unit is also upgradeable and can be upgraded with a standard 5V/1A power supply.

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HIGH QUALITY VOICE

๗5

This device is designed to provide high quality voice output for training purposes only. It is not intended for clinical use.

This device is designed to provide high quality voice output for training purposes only. It is not intended for clinical use.

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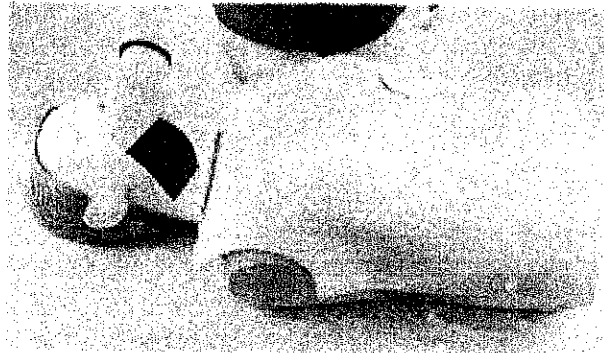
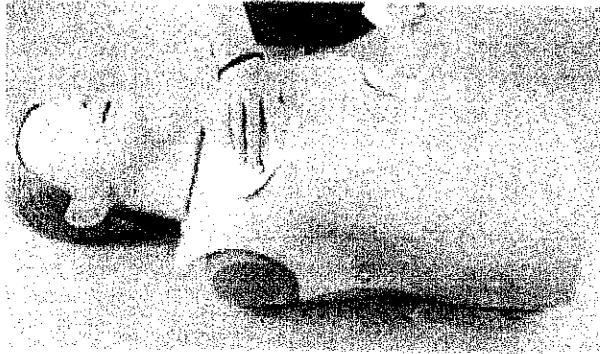
J6

AFFORDABLE REPLACEMENT PADS

Workshop showing the use of the J6 replacement pads.

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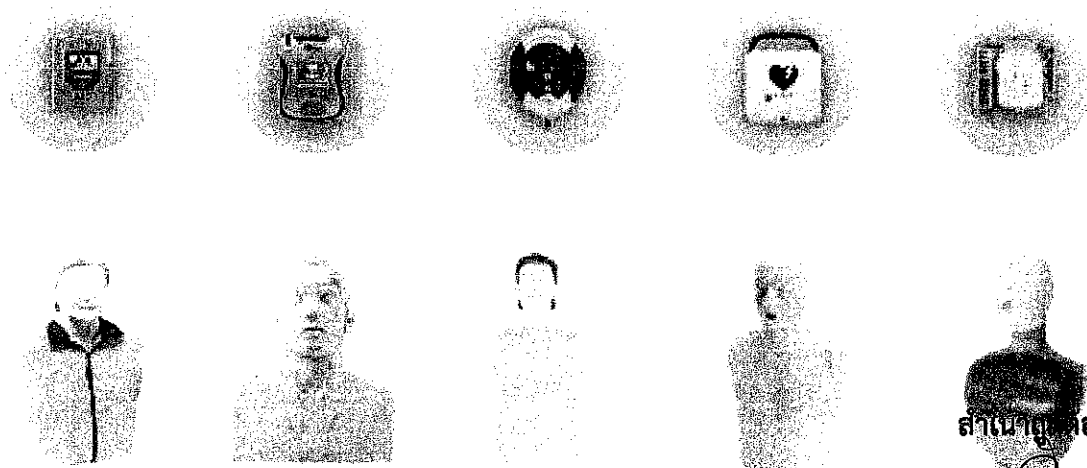
JB

MORE REALISTIC

On average, it takes 10-15 years to develop a new brand fragrance.

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To Whom It May Concern,

We, **Shenzhen WAP-health Technology Co., Ltd**, hereby confirm that **ASCENT CORPORATION CO., LTD.**, located in Thailand (214/201 Moo 5, Mae Hia Subdistrict, Mueang Chiang Mai District, Chiang Mai Province 50100, Thailand), is our:

- **Official Distributor**
- **Authorized Service Provider**

for AED Trainer ATM-112 within the territory of Thailand.

ASCENT CORPORATION CO., LTD. is authorized to promote, distribute, sell, and provide after-sales service and technical support for our products in the Thailand market.

This authorization is valid from 2026.07.31 to 2027.07.31, unless otherwise terminated or renewed by mutual agreement.

Should you require any further information, please feel free to contact us directly.

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ADD: Floor 3, Building C, The Third Industrial Zone #23, Gongming, Guangming New District, Shenzhen, Guangdong, China (Postal code 518106)

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